

DipMizer™ and Cassette DipMizer™

[DipMizer™](#)

[Cassette DipMizer™](#)

[My question has not been covered – how can I get more information?](#)

DipMizer™

[Can I use a DipMizer™ with any type of dip cup?](#)

[How should I clean my DipMizer™?](#)

[How often do I need to clean my DipMizer™?](#)

[How can I change the brush insert?](#)

[How often should I change the brush insert?](#)

[Is there a risk of bacterial build up?](#)

[Cassette DipMizer™](#)

[My question has not been covered – how can I get more information?](#)

- [Can I use a DipMizer™ with any type of dip cup?](#)

The DipMizer™ can be used with most Ambic teat dipping cups. However, it is not suitable for use with Foaming Dip Cups.

[Top](#)

- [How should I clean my DipMizer™?](#)

We recommend that your DipMizer™ is cleaned after every milking and dismantled for further cleaning once a week. Use a fresh, clean brush attachment for each milking.


After each milking the DipMizer™ can be cleaned by immersing it in hot water containing a detergent or sterilising agent. Massage the bristles of the brush well whilst in the water. Remove any dirt or congealed teat dip chemical from the brush. Allow the brush to dry thoroughly before putting it back onto the dip cup.

Once a week the brush assembly should be removed from the rubber holder by lifting the edge of the rubber around the steel rim of the brush. Both the rubber section and the brush should be cleaned (as with daily cleaning) to avoid build-up of dirt/chemical between the rubber and the steel.

[Top](#)

- How can I change the brush insert?

Changing the brush insert is very straightforward. Simply lift the edge of the rubber around the steel rim of the brush and remove (as recommended for weekly cleaning). Put the new insert in and let the rubber holder drop back down into position. The brush insert doesn't have a top or bottom, so it cannot be put in upside down.

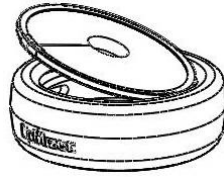

[Top](#)

- How often do I need to clean my DipMizer™?

The DipMizer™ should be cleaned after each milking and then completely dismantled for a deeper clean once a week.

[Top](#)

- How often should I change the brush insert?

We don't have a precise recommendation for this. However, if the brushes are visibly damaged, or are difficult to clean then they should be replaced. Chemical consumption will increase when the DipMizer™ starts to lose its efficiency.

[Top](#)

- Is there a risk of bacterial build up?

Failure to follow the recommended cleaning procedure may result in bacterial contamination of the teat dip chemical. Please see the related paper and poster for further information:

[Comparison of bacterial load between teat dip cup types with or without attachment](#)

[Top](#)

[Top](#)

Cassette DipMizer™

[Can I use a Cassette DipMizer™ with any type of dip cup?](#)

[How should I clean my Cassette DipMizer™?](#)

[How often do I need to clean my Cassette DipMizer™?](#)

[How can I change the brush insert?](#)

[How often should I change the brush insert?](#)

[Is there a risk of bacterial build up?](#)

[DipMizer™](#)

[My question has not been covered – how can I get more information?](#)

[Top](#)

- [Can I use a Cassette DipMizer™ with any type of dip cup?](#)

The Cassette DipMizer™ can be used with most Ambic teat dipping cups. However, it is not suitable for use with Foaming Dip Cups.

[Top](#)

- [How should I clean my Cassette DipMizer™](#)

We recommend that your Cassette DipMizer™ is cleaned after every milking and completely dismantled for further cleaning once a week. Use a fresh, clean brush attachment for each milking.

After each milking the Cassette DipMizer™ can be cleaned by immersing it in hot water containing a detergent or sterilising agent. Massage the bristles of the brush well whilst in the water. Remove any dirt or congealed teat dip chemical from the brush. Allow the brush to dry thoroughly before putting it back onto the dip cup.

Once a week the brush assembly should be completely removed from the cassette by pulling the straight edge of the plastic and unclipping the brush from its plastic holder. All parts of the unit should be cleaned (as with daily cleaning) to avoid build-up of dirt/chemical between the rubber and the steel. The brush insert doesn't have a top or bottom, so it cannot be put in upside down.

[Top](#)


- [How often do I need to clean my Cassette DipMizer™?](#)

The Cassette DipMizer™ should be cleaned after each milking and then completely dismantled for a deeper clean once a week.

[Top](#)

- How can I change the brush insert?

Changing the brush insert is very straightforward. Simply, remove the brush assembly from the cassette by pulling the straight edge of the plastic and unclipping the brush from its plastic holder. The brush insert doesn't have a top or bottom, so nothing needs to be lined up.

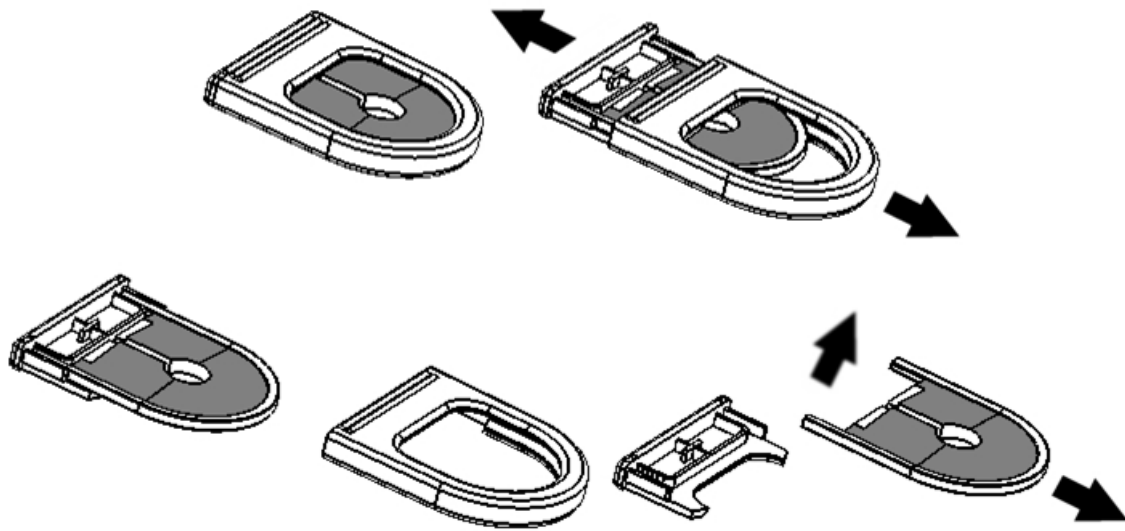

[Top](#)

- How often should I change the brush insert?

We don't have a precise recommendation for this. However, if the brushes are visibly damaged, or are difficult to clean then they should be replaced. Chemical consumption will increase when the DipMizer™ starts to lose its efficiency.

[Top](#)

- Is there a risk of bacterial build up?

Failure to follow the recommended cleaning procedure may result in bacterial contamination of the teat dip chemical. Please see the related paper and poster for further information:

[Comparison of bacterial load between teat dip cup types with or without attachment](#)

[Top](#)

- My question has not been covered – how can I get more information?

For further technical information please contact Ambic either by email or by telephone:

tech@ambic.co.uk

Tel: +44 (0) 1993 776555

[Top](#)

[Top](#)