LOCATION & FITTING (1 & 2)

The Mastitis Detector must be positioned in the long milk tube, between the Claw and the Milk Transfer line, and sited where it is easily seen and unlikely to collect excessive dirt or to be damaged. **IMPORTANT:** Use a section of the tube which empties of milk at the end of milking (see WARNING under CLEANING below regarding drainage).

IMPORTANT: when fitting to a high level or swing-over type parlour, the Mastitis Detector must be positioned as shown in illustration 1.

Mastitis Detectors will fit tubing of 12 - 14mm internal bore and should be fitted to all milking points. Cut the long milk tube at a suitable position. Clean and rinse the Mastitis Detector then connect into the tube by push fit. Lubricate or soften the tube in soapy hot water for easier connection. The milk Inlet with Arrow (see illustration 2) MUST point in direction of flow and be on the side facing the operator.

OPERATION The Mastitis Detector is designed for use and cleaning under vacuum not pressure. Removal and replacement of the screen is by direct pull and push. The body clamp is self-locking.

CLEANING Upon completion of milking, remove screen, reverse it and wash with running water to remove any deposits and external soiling. The screen must be in place during CIP cleaning. The design provides access for cleaning fluid to all milk contact surfaces of the Mastitis Detector during normal circulation cleaning.

WARNING: Do NOT use a brush or scraper on the mesh. Do not use cleaning chemicals at concentrations greater than those recommended by the chemical manufacturer. Failure of the screen may result if left immersed in Hypochlorite or Acid solution between milkings – it is therefore recommended to ALWAYS DRAIN CLEANING SOLUTION from detectors after CIP cleaning.

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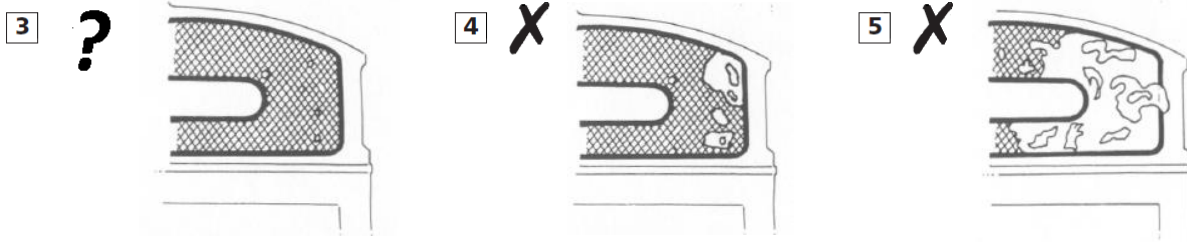
OPERATION

Clinical Mastitis is detected by the presence of clots on the downstream end of the screen shown in illustration 3.

Deposits will not normally accumulate along the length of the screen. The detector will not necessarily retain every clot but will show a representative sample. The milker must visually inspect each individual Mastitis Detector when the cluster is removed.

Illustrations 3 - 5 are intended as a guide to the type of conditions, which may arise. The quantity of clots indicates the degree of abnormality. A few clots (Fig. 3) found only once does not necessarily indicate Mastitis. However, repeated detection of a few clots, or many clots (Fig. 5), may indicate Mastitis requiring therapy under veterinary guidance. The affected quarter is found by palpation or by examination of the milk.

The Mastitis Detector is ergonomically designed for the rapid removal of mastitic milk deposits during milking. Simply remove the soiled screen, wash with running water, reverse it and replace. If a screen becomes soiled with deposits while other cows have yet to be milked, wait until the animal has finished milking then replace the screen. This way it is unnecessary to delay the milking routine while cleaning a dirty screen which can instead be cleaned afterwards.

NOTE: Any distortion observed in the mesh is due to the natural movement of plastics in conjunction with hot and cold liquids and does not in any way affect the performance of the unit.

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