## OPERATION

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## GB INSTALLATION

The Power Foamer is designed to be installed as a ring main system (Fig 1). The enclosure housing the electrics of the Power Foamer is rated at IP40 (IEC 60529), the power unit (A) must therefore be positioned outside of the milking parlour in a dry location and close to a 220-240V AC power socket. The power supply cord MUST remain accessible when the unit has been installed. Routing the ring main tube (B) will depend on the parlour design and will be either high level (Fig 2) or low level (Fig 3). If high level it will ideally be positioned above the rump rail with the long tail of the coil (D) connected via a 'T' fitting (C) and fixed to a suitable upright with cable ties (H). These will also be used to secure the ring main.

Alternatively it can be routed along the centre of the parlour.

CAUTION: Do not over tighten cable ties as air flow may be restricted.

'T' fitting and gun entry fitting are "push-in" and can be released by holding the flange in and pulling out the tubing.

Foam Applicators should be positioned such that all milking points can be reached comfortably without over-stretching the coils. Extension kits are available to enable as many Applicators as required to be installed.

If no suitable horizontal bar is available on which to hang the unit, it may be necessary to attach brackets (not included), or a hanging rail.

## OPERATION

Switch on at electric socket. Power Unit (A) will be activated. When a line pressure of 3psi. is reached, air pump will switch off via a pressure switch and will not operate until a gun is used and pressure drops. Fill bottle (G) with suitable foaming teat product to the 300ml mark - do not overfill. Bottle can either be unscrewed from Applicator (F) with gun (E:) attached or after releasing from the gun bayonet (Fig 4). When ready to dip, depress the trigger until foam is level or just above the rim of Applicator (F) and raise fully onto teat (Fig 5).

When milking is completed switch off electric power supply.

## MAINTENANCE

Use ONLY a soft dry cloth to clean the electrical enclosure housing when necessary - NEVER use a hose. If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard. Regularly clean the dip cups by detaching from gun, unscrewing the bottle then rinsing the cup out with clean warm water to remove any hairs, dirt, etc.

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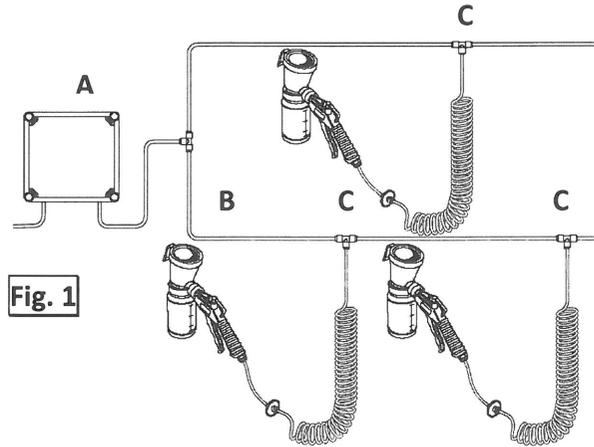
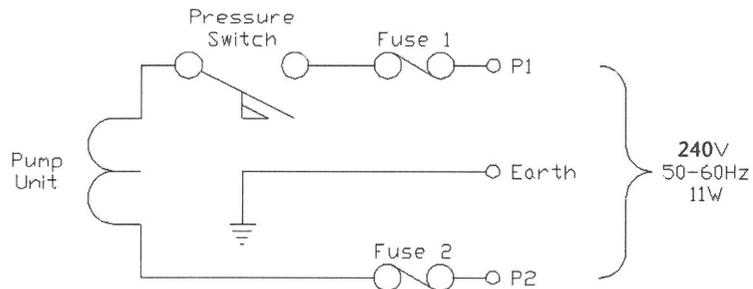
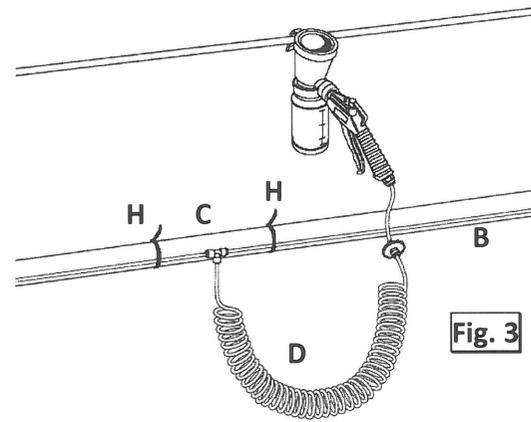
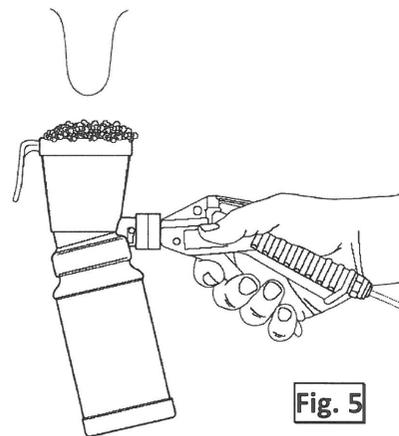
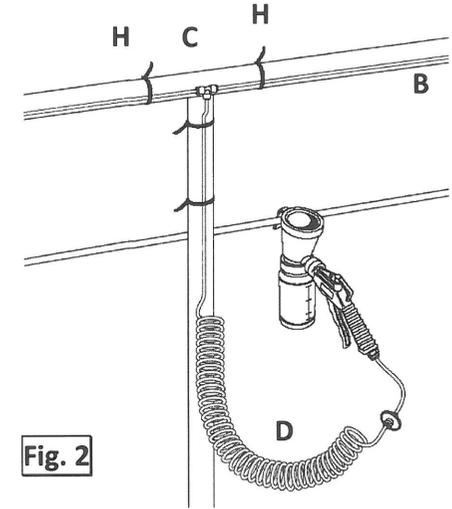
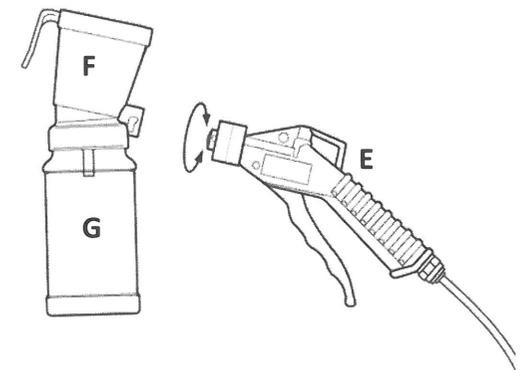
GB

### EC DECLARATION OF CONFORMITY

Name of Product: Electric Power Foamer  
 Type: APF/001 (APF/009)  
 Other identifying data: Serial Number  
 The Product complies with requirements of the following directives:

89/336/EEC  
 73/23/EEC

Harmonized standards which have been used:

EN 61000-6-3: 2001 BS EN 60335-1: 2002  
 EN 61000-6-1: 2001 BS EN 60335-2: 2002  
 EN 61000-3-2: 2000 EN 60335-1: 2003  
 EN 61000-3-3: 1995

Date: 21.12.04 Signed:

Name: R.J. Hiley  
 Position: Product Manager

Name and address of manufacturer: Ambic Equipment Ltd. Witney, Oxford OX28 4YF. ENGLAND  
 Name and address of agent: (when applicable)

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Nom et adresse du constructeur: Ambic Equipment Ltd. Witney, Oxford OX28 4YF. ENGLAND